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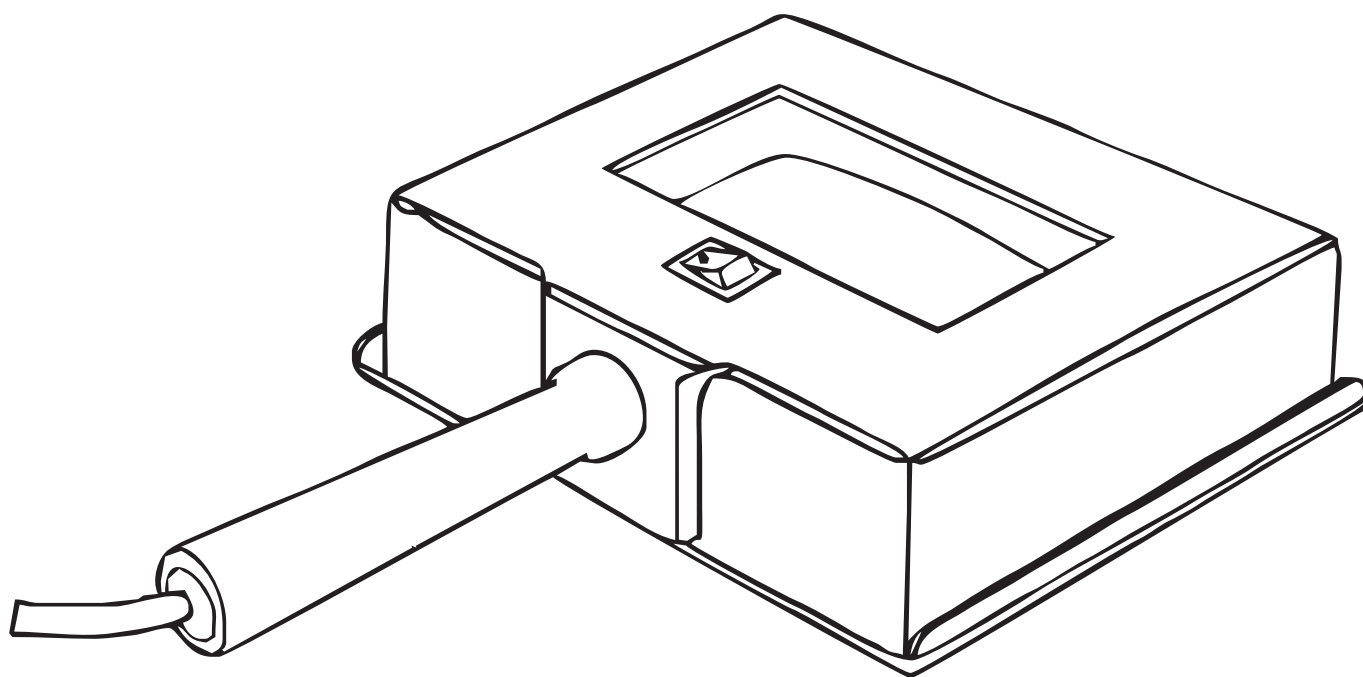
Skin Analyzer Lamp

Lampe analyseur de peau

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Introduction to Magnifier Lamp

Description - Skin Analyzer LAMP

The portable skin analyzer lamp is used by the esthetician to help analyze skin conditions. Different substances become luminous when exposed to the deep violet rays of the analyzer's lamp analyze the surface and deeper layers of skin in a totally dark room. The violet rays enable then esthetician to various skin conditions. Following are some example of skin conditions and how they appear under the analyzer's lamp.

How To Use

- 1) Clean the protective cover and lens; When cleaning, please ensure that the cleaning agent used will not damage the paint and plastics and ensure the maximum photometric output.
- 2) Before use, apply a massage brush to clean the skin surface.
- 3) Connect the power cord of analyzer lamp to the power supply, press the switch and the light will be on after a few seconds.
- 4) When using analyzer lamp, the instrument should be kept at a distance of 5-6 cm from the skin surface of the patient.
- 5) Wheck through a magnifying glass in a dark room.

Note: analyzer lamp are not allowed to overheat and avoid direct contact with the tube and skin.

Analyzer's LAMP Presents Skin Condition

Analyzer's lamp Indication	Shin Condition
Blue white	Normal and healthy skin
Bright fluorescent	Hydrated skin
Purple fluorescent	Thin skin without enough moisture
Light violet	Dehydrated skin

Analyzer's lamp Indication	Shin Condition
Yellow or sometimes Pink	Oily areas of the face and come done
Brown	Pigmentation and dark spots
White spots	Horniness layer of the skin and dead cells
White fluorescence	Thick conium layer

Non-guaranteed coverage

- Use by accident or error and damage caused by human activity
- Improper repairs by unauthorized personnel and unauthorized modifications caused faults
- Damage and failure caused by force majeure natural disasters

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