Lámpara analizadora de piel / Skin analyzer lamp / Lampe d'analyse de la peau

ES

ESTRUCTURA

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EN

STRUCTURE

1. The portable skin analyzer lamp is designed to help assess skin conditions. Different substances become visible or fluorescent when exposed to intense ultraviolet light, allowing examination of both the surface and deeper layers of the skin.
2. Made of high-strength ABS plastic.
3. Equipped with 4 UV light bulbs.

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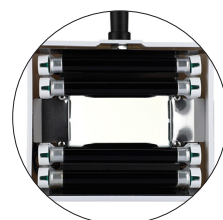
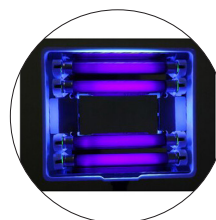
Analyzer's LAMP Presents Skin Condition

Analyzer's lamp Indication	Shin Condition	Analyzer's lamp Indication	Shin Condition
Blue white	Normal and healthy skin	Yellow or sometimes Pink	Oily areas of the face and come done
Bright fluorescent	Hydrated skin	Brown	Pigmentation and dark spots
Purple fluorescent	Thin skin without enough moisture	White spots	Horniness layer of the skin and dead cells
Light violet	Dehydrated skin	White fluorescence	Thick conium layer

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EN

INFORMATION

Read all instructions before beginning assembly. Keep the instructions for future use. Indoor use.

WARNING

Use only as intended.

ASSEMBLY

Follow the attached assembly instructions.

MAINTENANCE

Periodic maintenance tasks should be performed by checking the condition of the product. Ensure the product is in good condition and functioning properly. Check for breaks regularly. Remove the product from service when any condition that could be considered unsafe occurs.

CARE AND CLEANING

Clean the unit with a damp cloth or sponge using warm water and mild soap. Dry thoroughly. Do not use abrasive cleaners, bleach, chemical solvents, or strong detergents.

WARRANTY

Please refer to the General Terms and Conditions of Sale

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