
Instrucciones de operación
Operation instruction
Mode d'emploi

NUA

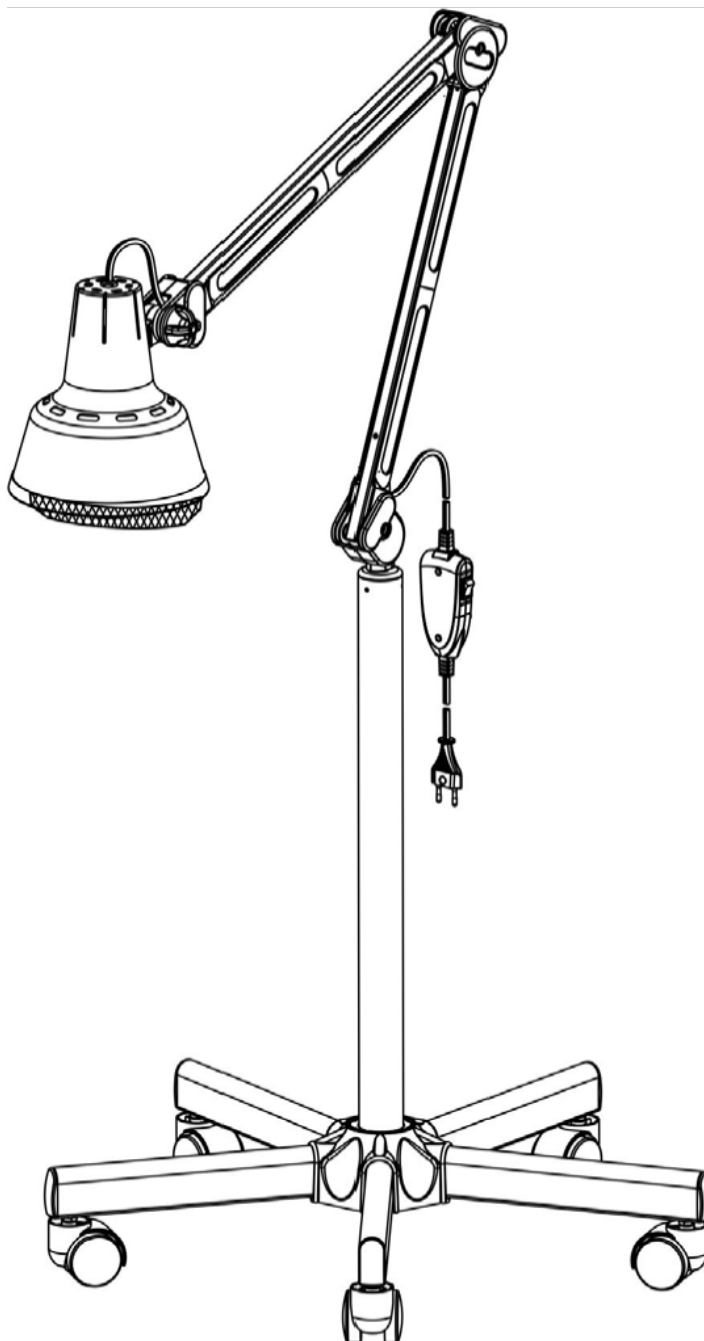
Welcome to **NUA** infrared therapy lamp!
Before using this infrared therapy lamp, please read
the operation instruction carefully.

H6102 Infrared therapy lamp Operating instruction

Features:

Infrared physiotherapy lamp is visible infrared lamp, Can penetrate into the characteristics of human tissues and infrared thermo-thermal effect, Increase tissue temperature expansion. Capillaries promote blood circulation, enhance metabolism, and improve tissue cell viability and regeneration. Infrared treatment of chronic inflammation can improve blood circulation, increase cell phagocytic function, eliminate swelling, and promote inflammation dissipation. The infrared ray is also used in the treatment of torsion and contusion, promoting tissue swelling and hematoma dissipation and reducing adhesion after surgery, promoting the softening of scar and reducing scar contracture.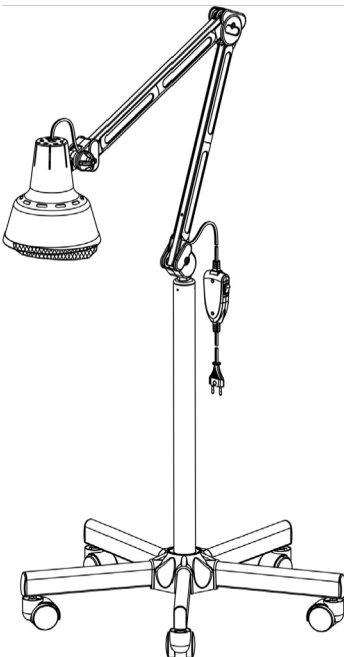


1. With its good thermal permeation, it can obviously relieve the pain of inflammation and blood circulation.
2. It has high radiation frequency, strong permeability, used for local infra-red radiation, improved metabolism, sterilization and accelerated wound healing.
3. It has obvious effect on muscle strain, joint pain, lumbar pain and cervical pain. Improve microcirculation, regulate nerve and endocrine function.
4. Improve blood flow rate, promote blood circulation, eliminate blood circulation barrier.
5. There are no obvious side effects, which can be combined with other therapies, better curative effect and shorter course of treatment. It is suitable for the penetration and absorption of the effective ingredients for the family, hospital, health center, physical therapy center, and beauty agency.
6. The product's bracket is protected from the finger. Novel line control design, feel good, easy to use.



Specification and technical parameters:

Product Name	Infrared therapy lamp	
model	H6102	
work voltage	AC 100~120V	AC 220~240V


Packing list:

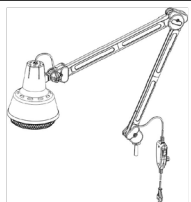Item	Product Description	QTY
Infrared therapy lamp		1 set
Column		1 PCS

power	150W	153W
work frequency	50/60Hz	
Product size	diameter 652mm high 1191mm	
N.W	7.6 Kg	

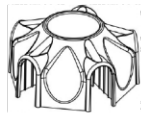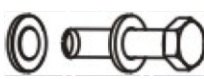
Five Feet Base		5 PCS
Five Feet Base		1 PCS
Five Feet Plat Broad		1 PCS
M10 screws		1set
M6 screws		5pcs
wrench		1 pcs
fixing sleeve		1 pcs
Instruction Manual		1 copy

Description: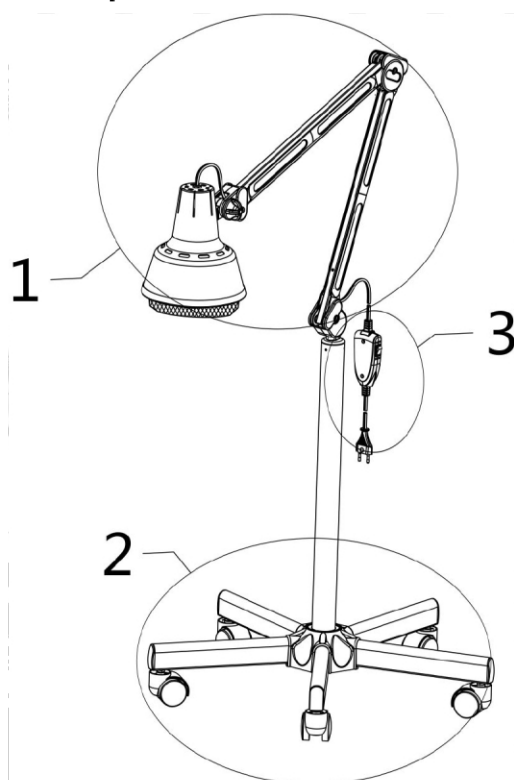

1. Infrared therapy lamp 2. Standing 3. console


Infrared therapy lamp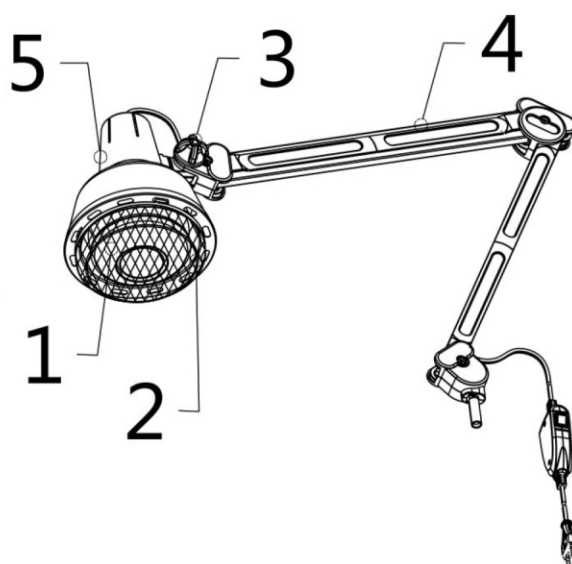

1. Infrared bulb
2. Protect lampshade
3. Adjusting knob
4. holder
5. Double-layer high temperature enclosure





Standing

1. column
2. fixing sleeve
3. Five Feet Base
4. Five Feet Plat Broad
5. M6 screws
6. M10 screws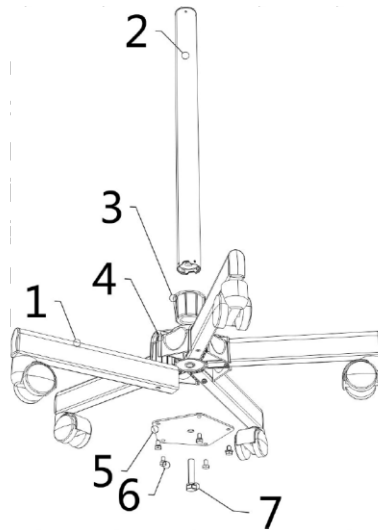

Console

1. power switch
2. Brightness adjustment knob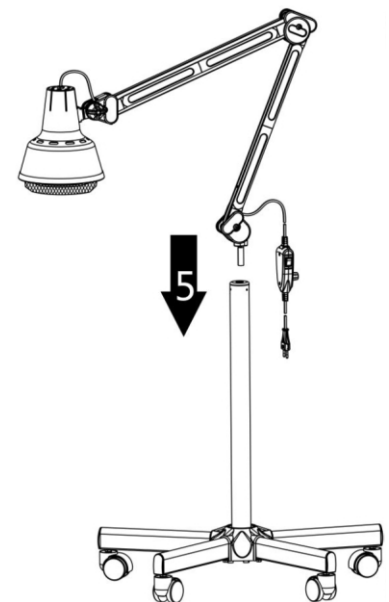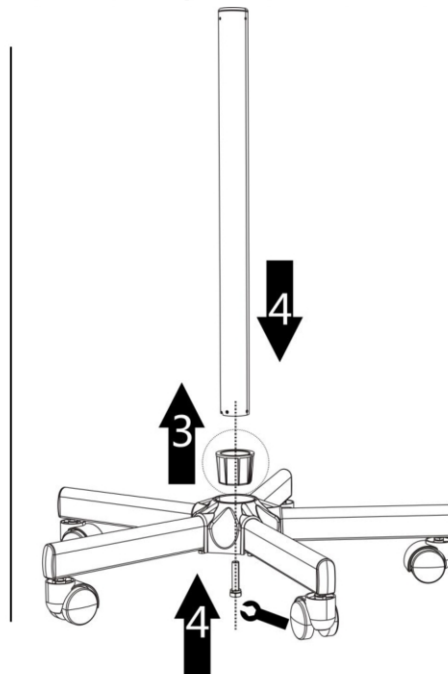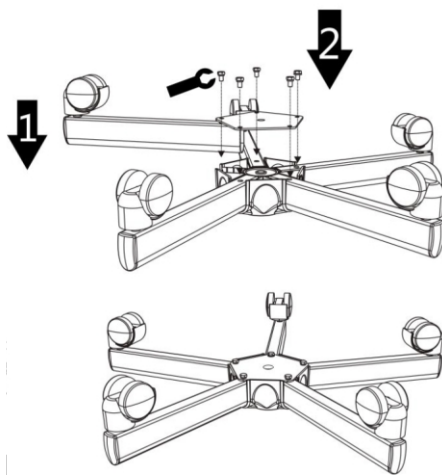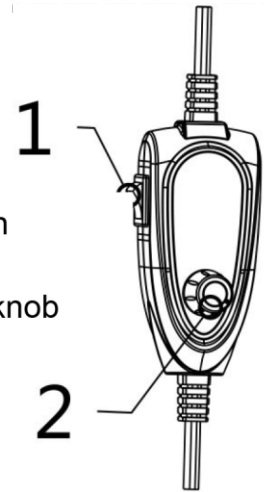

Assembly Instruction:

1. First, Press the five feet cover to the five feet base, and the Five Feet Base are pressed, and the Five Feet Base are pressed and the Five Feet Base are pressed together with five M6 Screws.


2. Then Set the column combine with the fixing sleeve in the five feet base (note: make sure to insert the fixed sleeve before inserting the Five Feet Base; otherwise, the Five Feet Base will not be inserted); Use the M10 Screws to cover Five Feet Plat Broad and the spring plate, and fasten the five feet base to the Column.

3. Put the infrared therapy lamp on the Column.

Operation instructions:

1. Plug the power plug;

2. Adjust the lamp to the suitable position, the lamp can be adjusted in multiple directions (as shown in the right picture), if the lamp is too tight or too loose, you can turn the knob to adjust the angle. Turn on the power switch on the console, the bulb can be used with visible red light, and the brightness adjustment knob is used to adjust the brightness. (As shown in figure)

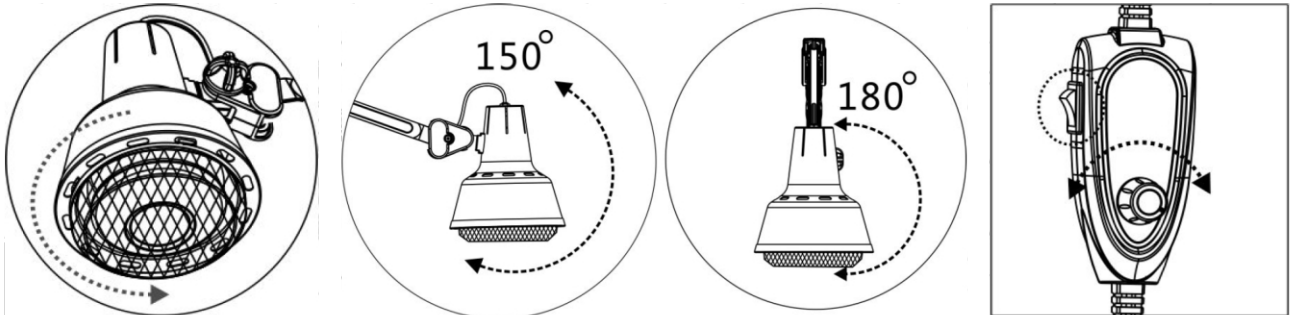3. turn off the power switch and rotate the knob to the minimum position in the end

Daily Maintenance:

1. When light bulbs appear dim without light or broken, please buy the same specifications conform of infrared light bulb replacement, To change the damaged connecting cord in appointed maintenance center. installing the light bulb will protect the chimney by counter-clockwise first and then take out (as shown) screw the bulb out, after changing the light bulbs, then gently rotate clockwise to protect chimney, tighten.
2. Adjust the rotation of the lamp head be careful, otherwise it will damage the joint of the lamp head.
3. During setting or move lamp, don't let fingers in the middle of the upper and lower arms for avoiding clamp. (As picture shown)
4. Turn off the power after finished. The lamp bulb is still hot. It should be placed for 5 minutes or so until the lamp be cooled and then collected.
5. Please place in places where children cannot touch and avoid danger.
6. If not used for a long time, please cut off the power supply, and put it properly after cleaning.

Warning!

1. It's **not** allowed to touch the high heat part, such as lamp shade, during the process of use, otherwise will be a danger of scald.
2. It's **not** allowed to place this machine under over humidity environment, like bathroom act.
3. If there's any exceptional problem, please contact to agency call professional maintainer to repair.
4. If power cord is destroyed, it's better to choose the accessories from the original factory. It's not allowed to repair by un-professional person and go to appointed repair agency to change.

If the picture on owner manual is different from the current product, then pls subject to current product. Our company will keep our right to improve .Pls forgive that the improvement would not notify if it is not affect normal use

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EN

IMPORTANT SAFETY INSTRUCTIONS

The product's cable cannot be stretched, tightened, or torn, because the product may stop working.

Do not shake the product, do not hit or the product may tip over and fall on users; causing damage to the product and/or injury to the Users.

The voltage of the product is 110v/220v, please choose the appropriate voltage to avoid burning the electrical parts.

When using the product, the maximum current can reach 10A or higher. Use high-power connections.

If possible, use an individual plug, to avoid power failures or fire due to overload.

If the cable is damaged, change it or contact the supplier for after-sales service.

Do not place fuels near the product.

Do not place objects of a different color than the surface, they may stain the skin of the product.

Do not spray water to clean the product, otherwise, it may cause product leakage, short circuits and may lead to electric shock or damage to the product. And that it stops working.

Do not expose the product to the sun, otherwise the color of the product may vary.

Unplug the product when it is not going to be used for a long period of time.

PRODUCT MAINTENANCE

Turn off the power off when product is not in use.

Do not stack sundries above the product, do not use sharp utensils access products.

The product must be placed in the dry environment instead of humid.

The product can not withstand the impact, so do not place heavy objects over 100Kg above the product.

The limit load of the product is 200Kg, using the product in safety load of 150Kg.

When the product does not work, please contact the supplier!

DAILY MAINTENANCE

After each session, use neutral reagents to remove dust and stains from the leather and wooden parts to prevent premature aging of the leather and wooden parts.

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EN

SUGGESTED USE

Before connecting, operating or adjusting this electric beauty bed, carefully read the following instructions and keep this manual safe.

This electric beauty bed uses a three wire power cable. When using it, it must be plugged into a three hole power socket with a ground wire to ensure that the electric beauty bed is properly grounded to prevent accidental electric shock for users, and to ensure the normal operation of the electric beauty bed.

When performing the direct control of the beauty bed's motor lifting operation, the control button must be continuously pressed until the desired position is released.

When using the electric push rod of this electric beauty bed, it should be noted that the frequency of use is Max 6min/h (Continuous operation within 1 hour cannot exceed 6 minutes)

Repairs must be carried out by trained professionals.

After each nursing session, use neutral reagents to remove dust and stains from the skin and decorative parts.

CLEANING OF THE PRODUCTS

Cut off power when cleaning product.

When cleaning product, remember to remove other item in product to prevent slipping down during the cleaning process, resulting in injuries and damaged goods;

Do not spray water to clean the product, otherwise, it may produce a short circuit, or may produce some electric shock.

Use a special cleaning agent when cleaning, oily liquid can not be used, or will cause the leather contamination, corrosion and other phenomena.

Do not use cleaning agent with a liquid dye when cleaning, or will cause the damaged to the local color of the leather;

Do not use cleaning agent with corrosive liquid, or will cause the damaged to the plastic and leather.

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EN

Our products are designed and manufactured according to the applicable regulations.

INFORMATION

Read all instructions before beginning assembly. Keep the instructions for future use.

WARNING

Use only as intended. Do not attempt to stand or kneel on the item. Do not stack. Only flat combinations.

ASSEMBLY

See assembly instructions.

CARE AND CLEANING

Clean the unit with a damp cloth or sponge using warm water and mild soap. Dry thoroughly. Do not use abrasive cleaners, bleach, chemical solvents, or strong detergents.

WARRANTY

2-year limited warranty.

MAINTENANCE

Periodic maintenance tasks should be performed by checking the condition of the product. Ensure the product is in good condition and functioning properly. Check for breaks regularly. Remove the product from service when any condition that could be considered unsafe occurs.

With this guide, you can keep your products in the best possible condition and extend their lifespan. Good luck with your salon maintenance!

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For possible future updates to the document, visit this QR link to see the latest version

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