H2301

Maia High

High Frequency / Alta Frecuencia / Haute fréquence / Alta frequenza / Hochfrequenz

EN	2
ES	5
FR	8
IT	11
DE	14

DDUUEETT EUROPE would like to thank you for choosing Sonia products. We hope that this instruction manual will help you get the most out of this product.

If you have any doubts regarding its use and/or operation, please contact your distributor.

BEFORE USE

- Please read the entire instruction manual carefully before using the product. Take note of any warnings included in this manual.
- Always keep this instruction manual handy and if you give the device to another person, hand over the manual as it is an essential part of the product.
- Some components of the product may differ from the image that appears in catalogues, websites, brochures, etc.
- In order to enhance the product, the manufacturer reserves the right to modify its features and/or components without prior notice.

CORRECT USE OF THE DEVICE

- This product has been designed to be used by beauty professionals at beauty salons and should not be used for other purposes.
- Please note that any misuse of the device entails the loss of the guarantee.
- Do not make any modification to the device without the authorisation of DDUUEETT EUROPE SL.
- DDUUEETT EUROPE SL shall not be held liable for any material or personal damage due to the misuse of the product.

CONTRAINDICATIONS

- This product should not be used on:
 - People with metal implants in their body.
 - People with pacemakers or heart problems.
 - Pregnant women.
 - People with sensitive skin, lesions, inflammations, or any skin problem or disease.
- With people undergoing medical treatment or a diagnostic process, the product must be used with medical authorisation.

PRECAUTIONS

- Before using it for the first time and after each use, check the device and the mains cable for any damage.
- Unplug the device from the mains before starting to clean it or undertaking any type of maintenance.
- Do not use the product outdoors or in places where it may be affected by humidity or inclement

weather.

- If the power supply cable is defective, unplug the appliance immediately and inform your dealer or specialised technician.
- Position the power supply cable properly so that people cannot trip over it and make sure you plug it into an earthed socket.
- At the end of the working day, disconnect the device and unplug the power supply cable from the mains.
- If you detect any anomaly in the operation of the product, please contact your distributor or specialised technician.

SAFETY INDICATIONS

- The device and accessories should be kept out of the reach of children.
- Only plug the device into a properly installed socket. The local mains voltage must correspond to the technical specifications of the device.
- The power socket must be accessible in case you have to quickly unplug the device at any given time.
- Always pull out the actual plug and not the mains cable.
- Never tamper with any electrical component or open up/repair any component of the device.
- Only your distributor or specialised technician should repair the device.
- Do not switch on the device if there is visible damage to it or to the mains cable.
- Never touch the device or the mains cable with wet hands.

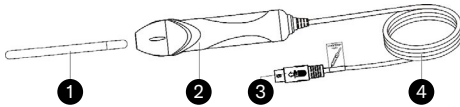
CONTENTS

- Main body x 1 unit
- Power cable x 1 unit
- High frequency handpiece x 1 unit
- Mushroom-shaped electrode x 1 unit
- Spoon-shaped electrode x 1 unit
- Ball-shaped electrode x 1 unit
- Straight electrode x 1 unit
- Instruction manual x 1 unit

TECHNICAL SPECIFICATIONS

Product	Alta frecuencia
Reference	H2301
Voltage	220-240 V
Input frequency	50/60 Hz
Power	20 W
Dimensions	28 x 17 x 11 cm
Net weight	2,25 Kg

COMPONENTS

- ① Electrode hose
- ② High frequency handpiece
- ③ High frequency cable connector
- ④ High frequency flexible cable

This device also incorporates four different types of glass electrodes:

- Mushroom-shaped electrode

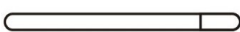
- Spoon-shaped electrode


- Ball-shaped electrode

- Straight electrode


CONTROL PANEL

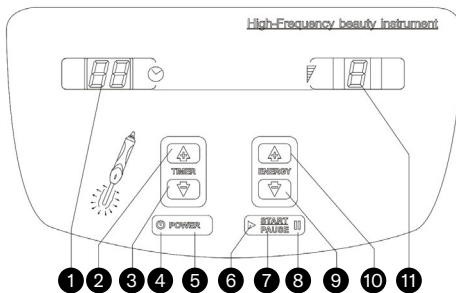- ① Timer
- ② Increase time button
- ③ Decrease time button
- ④ Power on indicator
- ⑤ On / off button
- ⑥ Run indicator
- ⑦ Start / Stop button


- ⑧ Pause indicator
- ⑨ Decrease power button
- ⑩ Increase power button
- ⑪ Power display

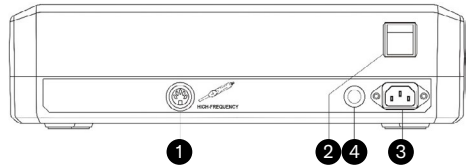REAR PANEL

- ① High frequency handpiece cable connection
- ② Power supply switch
- ③ Electrical power supply connector
- ④ Fuse

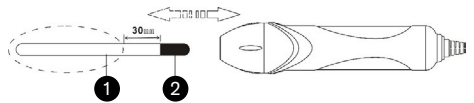
CONNECTION AND ASSEMBLY INSTRUCTIONS

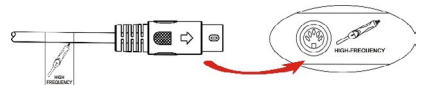- ① Glass part of the electrode
- ② Metal part of the electrode

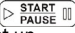
- During assembly and disassembly of the high frequency electrode, handle the area marked with a dashed line, keeping a distance of 30 mm from the metal part of the electrode.

- Connect the power cable to the rear panel of the main body.
- Connect the cable of the high frequency handpiece to the corresponding output on the rear panel.

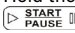
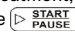
POWER AND CONFIGURATION

- Turn on the power switch on the rear of the machine and press [POWER] on the control panel. The power indicator will light up. The machine will be in standby mode and ready for operation. On the control panel:
 - The timer will show 15 minutes.
 - The power indicator will show 1.

- On the  button, the pause indicator  will light up.
- Set the timer using the  or  buttons to choose the operating time of the machine, in a range of between 1 and 30 minutes.
- Set the power using the  or  buttons to choose the power of the machine, in a range of between 1 and 9.
- Use the  button to start or stop the device.
 - When running, the  indicator will light up.
 - When paused or on standby, the  indicator will light up.

OPERATION

- Direct electrotherapy:
 - Select the appropriate electrode for the treatment, set the timer and decrease power to the minimum.
 - Hold the high frequency handpiece and press  until the run indicator  lights up.
 - Then bring the electrode closer to the area of the skin to be treated until it meets the skin and set the power  gradually.
 - At the end of the treatment, reduce the power  and press the  button until the pause indicator  lights up and the machine enters pause mode.
 - Then you can remove the electrode from the patient's face.
- Indirect electrotherapy:
 - The procedure is the same as above, except that the patient should hold the straight electrode in their hand from the beginning of the treatment.
 - The operator's electrode should not come into contact with the patient's electrode.
 - During the treatment, the operator should keep one hand on the patient's face.

Bear in mind:

- Humming sounds indicate that the high frequency electrode is working normally.
- If using direct electrotherapy, when the electrode is less than 2.5 cm away from the skin, a spark will occur. This is why you should set the power to the minimum and increase gradually. If the spark still occurs, you can use a dry cotton cloth to cover the face.
-  should be off when placing and removing the high frequency electrode.

- Before cleaning the device, unplug it from the mains.
- Clean the device with a soft, dry cloth. Do not use liquids to clean it.
- After each use, the various components and accessories must be sterilised and left to dry.
- For maintenance or repairs of any part of the product, please contact your distributor or specialised technician.

RECYCLING

If you need to dispose of the product or any of its parts or accessories, please check and respect the laws and regulations applicable to waste management and environmental protection.

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