H2101

Maia Ultra

Ultrasounds / Ultrasonidos / Ultrasons / Ultrasuoni / Ultraschall

EN	2
ES	5
FR	8
IT	11
DE	14

DDUUEETT EUROPE would like to thank you for choosing Sonia products. We hope that this instruction manual will help you get the most out of this product.

If you have any doubts regarding its use and/or operation, please contact your distributor.

BEFORE USE

- Please read the entire instruction manual carefully before using the product. Take note of any warnings included in this manual.
- Always keep this instruction manual handy and if you give the device to another person, hand over the manual as it is an essential part of the product.
- Some components of the product may differ from the image that appears in catalogues, websites, brochures, etc.
- In order to enhance the product, the manufacturer reserves the right to modify its features and/or components without prior notice.

CORRECT USE OF THE DEVICE

- This product has been designed to be used by beauty professionals at beauty salons and should not be used for other purposes.
- Please note that any misuse of the device entails the loss of the guarantee.
- Do not make any modification to the device without the authorisation of DDUUEETT EUROPE SL.
- DDUUEETT EUROPE SL shall not be held liable for any material or personal damage due to the misuse of the product.

CONTRAINdicATIONS

- This product should not be used on:
 - People with metal implants in their body.
 - People with pacemakers or heart problems.
 - Pregnant women.
 - People with sensitive skin, lesions, inflammations, or any skin problem or disease.
- With people undergoing medical treatment or a diagnostic process, the product must be used with medical authorisation.

PRECAUTIONS

- Before using it for the first time and after each use, check the device and the mains cable for any damage.
- Unplug the device from the mains before starting to clean it or undertaking any type of maintenance.
- Do not use the product outdoors or in places where it may be affected by humidity or inclement

weather.

- If the power supply cable is defective, unplug the appliance immediately and inform your dealer or specialised technician.
- Position the power supply cable properly so that people cannot trip over it and make sure you plug it into an earthed socket.
- At the end of the working day, disconnect the device and unplug the power supply cable from the mains.
- If you detect any anomaly in the operation of the product, please contact your distributor or specialised technician.

SAFETY INDICATIONS

- The device and accessories should be kept out of the reach of children.
- Only plug the device into a properly installed socket. The local mains voltage must correspond to the technical specifications of the device.
- The power socket must be accessible in case you have to quickly unplug the device at any given time.
- Always pull out the actual plug and not the mains cable.
- Never tamper with any electrical component or open up/repair any component of the device.
- Only your distributor or specialised technician should repair the device.
- Do not switch on the device if there is visible damage to it or to the mains cable.
- Never touch the device or the mains cable with wet hands.

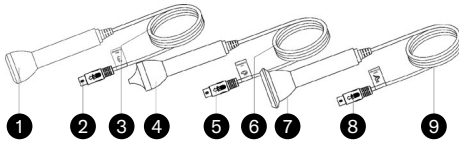
CONTENTS

- Main body x 1 unit
- Power cable x 1 unit
- Face probe x 1 unit
- Eye probe x 1 unit
- Body probe x 1 unit
- Instruction manual x 1 unit

TECHNICAL SPECIFICATIONS


Product	Ultrasounds
Reference	H2101
Voltage	85-265 V
Input frequency	50/60 Hz
Power	30 W
Ultrasound output frequency	> 1 MHz
Dimensions	28 x 17 x 11 cm
Net weight	2,35 Kg

COMPONENTS

- 1 Face probe
- 2 Face probe cable connector
- 3 Face probe flexible cable
- 4 Eye probe
- 5 Eye probe cable connector
- 6 Eye probe flexible cable
- 7 Body probe
- 8 Body probe cable connector
- 9 Body probe flexible cable

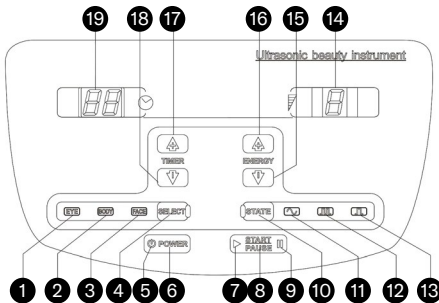
CONTROL PANEL

- 1 Eye probe run indicator
- 2 Body probe run indicator
- 3 Face probe run indicator
- 4 Probe selection
- 5 Indicator on
- 6 On / Off button
- 7 Start indicator
- 8 Start / Stop button
- 9 Pause indicator
- 10 Wave type selection
- 11 Continuous wave indicator
- 12 Discontinuous wave indicator
- 13 Pulse wave indicator
- 14 Power display
- 15 Decrease power button
- 16 Increase power button

- 17 Increase time button
- 18 Decrease time button
- 19 Timer

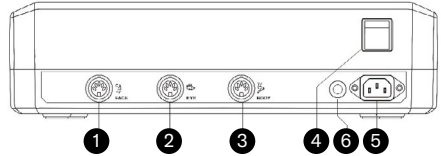
REAR PANEL

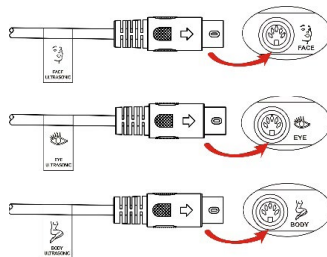- 1 Face probe cable connection
- 2 Eye probe cable connection
- 3 Body probe cable connection
- 4 Power supply switch
- 5 Power supply connector
- 6 Fuse


CONNECTION INSTRUCTIONS

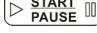- Connect the power cable to the rear panel of the main body.
- Connect the three probe connection cables to the corresponding outputs on the rear panel.

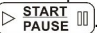
POWER AND CONFIGURATION

- Turn on the power switch on the rear of the machine and press **POWER** on the control panel. The power indicator will light up. The machine will be in standby mode and ready for operation. On the control panel:
 - The timer will show 15 minutes.
 - The power indicator will show 5.
 - The face probe indicator **FACE** will light up.
 - The continuous wave indicator will light up.
 - On the **START PAUSE** button, the pause indicator will light up.

- Set the timer using the  or  buttons to choose the operating time of the machine, in a range of between 1 and 30 minutes.
- Set the power using the  or  buttons to choose the power of the machine, in a range of between 1 and 9.
- Probe selection: use the  button to select the ultrasound probe to be used.
 - When the  indicator lights up, the face probe is selected.
 - When the  indicator lights up, the body probe is selected.
 - When the  indicator lights up, the eye probe is selected.
- Wave type selection: use the  button to select the type of probe.
 - When the  indicator lights up, the type of wave selected is continuous.
 - When the  indicator lights up, the type of wave selected is discontinuous.
 - When the  indicator lights up, the type of wave selected is pulse.
- Use the  button to start or stop the device.
 - When running, the  indicator will light up.
 - When paused or on standby, the  indicator will light up.

OPERATION

- Connect the device and leave it in standby mode with the  indicator lit.
- Thoroughly clean the area of skin to be treated with the appropriate product according to skin type and remove any remaining product which may prevent it from penetrating again when applying the ultrasound treatment.
- Apply conductive gel across the whole area to be treated.
- Select the probe to be used (face, body or eyes), the operating time and set the power to minimum.
- Hold the handpiece of the ultrasound probe and press . The run indicator  will light up.
- Massage the area to be treated with small circular movements for the previously selected treatment time, setting the desired power.

Bear in mind:

- Never use on dry skin. Always apply enough conductive gel for the entire treatment time.
- The treatment time should never be longer than

15 minutes and the power used should not be too high. Set the time and power according to the area of skin to be treated.

CLEANING AND MAINTENANCE

- Before cleaning the device, unplug it from the mains.
- Clean the device with a soft, dry cloth. Do not use liquids to clean it.
- After each use, the various components and accessories must be sterilised and left to dry.
- For maintenance or repairs of any part of the product, please contact your distributor or specialised technician.

RECYCLING

If you need to dispose of the product or any of its parts or accessories, please check and respect the laws and regulations applicable to waste management and environmental protection.

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