

Instructions Manual

Portable Massage Table

Nomad One

VERSION 1.26

WK *nomad one*

Open the Massage Table

1. Release the locking clasps and unfold the massage table as (Fig 1)
2. Extend the table legs, ensuring that the cables attached to the table leg structure is not tangled, and remove any accessories.(Fig 2)
3. Place your left foot against the table leg and use your right hand to lift the handle upward, securing the table on floor.(Fig 3)
4. Adjust the table's position to ensure the surface is completely level.(Fig 4)

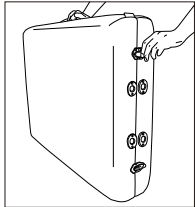


FIG.1

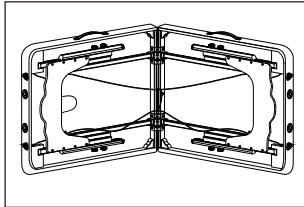


FIG.2

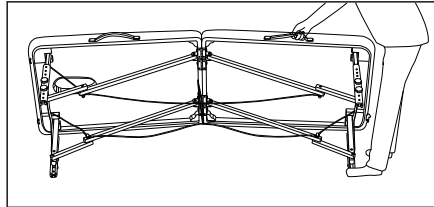


FIG.3


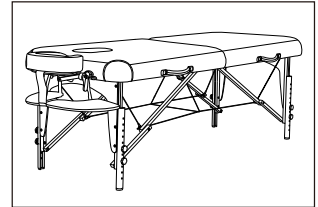


FIG.4

Height Adjustment

Your table's legs are adaptable. The table comes preset at a height comfortable for most individuals. To change the height, unlock the knobs on all four legs, detach the leg, position them at your desired height, and then re-lock the knobs.

Avoid excessive tightening of the knobs. Periodically checking their tightness is advised.

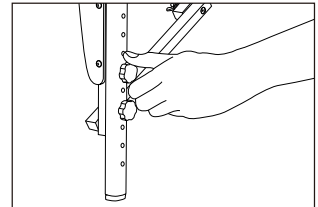

Leather Care

- Avoid exposing the leather mattress cover to humid environments, water, extreme cold, or high heat to prevent cracking.
- Use sheets to shield the bed surface and prevent sweat from damaging the leather. This helps extend the life of the leather surface.
- For cleaning, use a neutral detergent and ensure thorough drying after wiping.

Daily Maintenance Guidelines

- Regularly inspect the massage bed to ensure all screws are tightened and there are no visible cracks in the wooden structure.
- The maximum working load for the folding massage bed is 204kg/450lbs. Please do not exceed this weight limit as overloading may lead to structural damage and cause personal injury.
- To ensure stability and safety, advise users to lie in the center of the bed to evenly distribute pressure. Also, ensure the bed is placed on a sturdy surface and all four bed legs are at the same level.
- Safety and stability are essential for proper use and maintenance of the massage bed.
- If not in use for a short period, store the massage bed in the carrying bag and keep it in a dry, cool place away from direct sunlight.



- Remind:**
- Do not utilize alcohol-based, petroleum-based cleaners, or any alcohol-containing products.
 - Prevent contact with sharp or hard objects that could damage the leather surface.

WK

Imported by
WEELKO BARCELONA SL

Pol. Ind. La Valldan
Camí de Garreta, 92
08600 Berga
Barcelona - Spain
T. +34 93 824 90 20
E. info@weelko.com