EN

vibbel

0006871

Aty

Pedicure micromotor

Thanks for your purchase!

From DDUUEETT EUROPE SL we thank you for your confidence in buying Vibbel products. We trust that this instructions manual will help you to get the most out of this product.

We also remind you to contact your distributor for any questions regarding the use and/or operation of this product.

GENERAL DESCRIPTION OF THE PRODUCT AND EXPECTED USES

This product has been designed to be used by beauty professionals in beauty centres and must not be used for other purposes.

FIRST OF ALL...

- Read in detail this instructions manual before using the product.
- DDUUEETT EUROPE SL will not be responsible for any physical or material damage, accident or harm due to misuse of the product.
- Some components of the product may differ from the image that appears in catalogues, website, brochures, etc.
- In order to improve the product, the manufacturer reserves the right to modify the features and/or its components without prior notice.


TECHNICAL FEATURES OF THE PRODUCT

| | |
|---|--------------|
| Package dimensions (cm) | 27x23,5x9 |
| Package volume (m3) | 0,01 |
| Gross weight (kg) | 2 |
| Product dimensions (cm) | 15x13x8 |
| Diameter of the handle of the drill to be installed | 2,35 mm |
| Net weight (kg) | 1,7 |
| Input voltage | 110 ~ 240 V |
| Output voltage | 0 ~ 32 V |
| Frequency | 50 / 60Hz |
| Fuse | 2A (TA2/250) |
| Speed | 35.000 rpm |

INCLUDED IN THE PACKAGE:

- Unit control with power cable x 1 unit
- Micromotor handpiece x 1 unit
- Desktop support for handpiece x 1 unit
- Foot controller ON/OFF with connection cable x 1 unit
- Tool x 1 unit
- Fuse TA2/250 x 1 unit
- Set of 2 carbon brushes x 1 unit
- Instruction manual x 1 unit

RPM adjustment according to the diameter of the drill: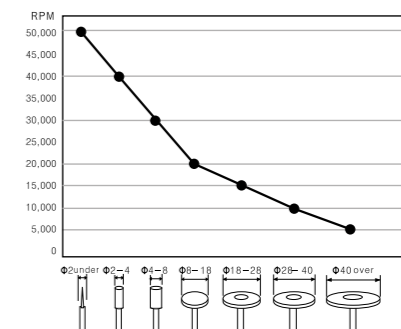


CERTIFICATIONS C E

The product you have purchased complies with the following European directives:

- 2014/30/EU Electromagnetic Compatibility
- 2014/35/EU Low Voltage Directive

CONNECTIONS AND OPERATION

Once all the components included in the package and its connections are identified:

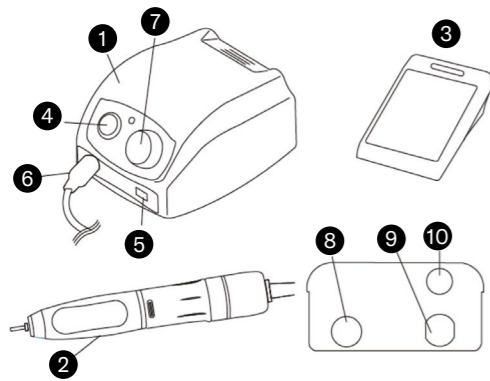- 1 Control unit
- 2 Micromotor handpiece
- 3 Foot controller ON/OFF
- 4 Switch: ONF "I" / OFF "O"
- 5 Rotation selection: Right "R" / Left "L"
- 6 Micromotor handpiece cable connection
- 7 Speed control "rpm"
- 8 Foot controller connection
- 9 Power supply cable
- 10 Fuse location

- Connect the foot controller connection cable 3 to the rear of the control unit 8.
- Connect the micromotor handpiece cable 2 to the front of the control unit 6.
- On the control unit 1, turn the switch to "O" 4 and set the speed to "0 rpm" 7.

Insert the drill (not included in this equipment) into the micromotor handpiece 2. To do this, follow the following steps:

- Hold the micromotor handpiece with your hand and note that in the central part there is a small movement towards one direction.
- Following the same direction of rotation, turn it firmly until you hear "click". With this movement the support of the drill will be free and can be removed.

- Replace the drill, so that its support is at least 1 cm. inside and it is correctly settled. The length of the protruding part of the support of the drill must not exceed 3,5 cm.
- Turn the moving part of the handpiece in the opposite direction to the previous point until you hear "click" again.

- Check that the drill is correctly settled by pulling it out and checking it does not move at all. - Plug the power cable of the control unit to the power supply 9.

Next, there are 2 ways to use the micromotor:

1. To use the micromotor without the foot controller:


- On the control unit 1, press the switch to the ON position "I" 4 and adjust the speed progressively to the desired "rpm" 7.
- To change the direction of rotation of the drill, press the switch to the OFF position "O" 4 and select the desired direction of rotation 5. Press again the switch to the ON position "I" 4 to start it up.
- To stop it, press the switch to the OFF position "O" 4.

2. To use the micromotor with the foot controller:

- On the control unit 1, press the switch to the OFF position "O" 4.
- Activate the foot controller 3.
- Adjust the speed progressively to the desired "rpm" 7.
- To change the direction of rotation of the drill, stop operating the foot controller 3 and select the desired direction of rotation 5. Reactivate the foot controller 3 to start it up.
- To stop it, stop operating the foot controller 3.

CLEANING AND MAINTENANCE

- To remove the dust, use a dry cloth.
- In case it is necessary to change the fuse: remove the cover 9, remove the fuse and replace it with one of the same characteristics (T2A/250V). Put the cover again 9.
- The brushes of micromotor must be periodically checked, since in case of being in bad conditions the equipment could be damaged.
- To change them, use the micromotor for 1 minute without any drill. Then, switch off the equipment and unplug from the power supply. Follow the 3 following steps:
 1. Unscrew the final part of the handpiece.
 2. Unscrew the 2 screws on each side.
 3. Replace the brushes with new ones, and then screw the final part of the handpiece again.

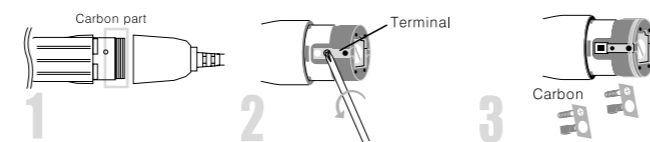SAFETY PRECAUTIONS

- Before connecting and starting up the product, read the user manual carefully.
- Unplug the product from power supply before starting the cleaning, any maintenance or changing the drill.
- Before plugging the product to the power supply, make

sure that the ON/OFF switch is in OFF position (O).

- Do not use the equipment outdoors or in any place where it may be affected by inclement weather or humidity.
- Do not use the product with a rusted, bend or damaged head or drills. All heads or drills must comply with the ISO standards that apply to them.
- Start using the device at low speed. Check that there are no vibrations or problems. Then, progressively increase to the desired "rpm" following the table "RPM adjustment according to the diameter of the drill".
- Do not drop the handpiece, as it could be damaged.
- If the power supply cable is damaged, unplug the device immediately and inform your distributor or specialized technician.
- When device is not in use, unplug from the power supply.
- If any anomaly in operation the product is detected, consult your distributor or specialized technician.
- Place the device where it cannot be reached by a child or unauthorized person.
- If you don't have a technical expert at your disposal for maintenance or repairing any part of the product, contact your distributor or specialized technician.

RECYCLING

If you have to get rid of any part or accessory of this product, please check the local and state laws that apply to waste management and environment.

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